





# AL200 Pen Replacement Instructions

Inside the AL200 Pen Replacement Kit Bag you will find:

A.) Pre-filled replacement marker pens (3) 	B.) Screws 	C.) Springs (3)* 
D.) Screwdriver 	E.) Instruction Card (1) <div style="border: 1px solid black; padding: 5px; text-align: center;">Place this card on Lens Table to prevent parts from falling inside Lensmeter.</div>	

Note: The screwdriver has a special attachment to hold screws for easier use.

\* Screws and Springs are spares. Existing screws and springs can be reused.

## Installing New Marker Pens

**NOTE:** Ink from the AL200 marker pens can be messy. Please consider using a tissue while handling old and new marker pens. (Tissue not supplied in kit)

- 1 Lower the Lens Table to its bottom position using the Table Height Lever.
- 2 Slide the Instruction Card onto the Lens Table to cover any gaps that would allow parts to fall into the AL200.
- 3 Slide the Marker Pens towards the Nosepiece using the Lens Marker Lever.
- 4 While holding the first Marker Pen at the bottom, unscrew the Marker Pen screw using the Screwdriver.
- 5 Slide the Marker Pen and spring out of the Lens Marker Holder.
- 6 Repeat Steps 3 - 5 for the remaining two Marker Pens.
- 7 Slide one Spring onto a Marker Pen and slide both into the Lens Marker Holder.
- 8 While holding the bottom of the Marker Pen, use the screwdriver to place and tighten the supplied screw.
- 9 Repeat Steps 7 & 8 for the remaining two pens.
- 10 Remove the Instruction Card from the Lens Table and discard the old Marker Pens.

