

Service Bulletin

REICHERT® NON-CONTACT TONOMETERS

SHIPPING PROTECTOR

The Reichert 7, Reichert 7CR, Ocular Response Analyzer®, and the Ocular Response Analyzer G3 come with a shipping protector installed on the patient's side of the instrument. This protector ensures that the motor does not get damaged during shipping. It is **EXTREMELY** important that this plastic shim is saved and installed anytime the unit is shipped to prevent damage to the motor and malfunction of the unit, most likely an M15 motor error.



CAUTION: IT IS VERY IMPORTANT TO REMOVE THE SHIPPING PROTECTOR BEFORE OPERATING THE UNIT.



1. Pull the plastic Shipping Protector up and out of the opening on patient side of the instrument.

IMPORTANT: Do **not** throw away the Shipping Protector! Keep it with the rest of the packaging material so that it can be re-installed if future shipment is necessary.

2. Continue with the instrument setup as indicated in the User's Guide.

Note: Anytime the unit is shipped, this plastic Shipping Protector must be reinstalled, to prevent any damage to the unit.

AMETEK[®]
ULTRA PRECISION TECHNOLOGIES

Reichert, Inc. 3362 Walden Avenue, Depew, NY 14043
Toll Free: 888-849-8955 • Tel: (716) 686-4500
Fax: (716) 686-4545 • www.reichert.com

Reichert[®]
TECHNOLOGIES

ISO 9001/13485 Certified
February 10, 2016
16050-105 Rev. C