

Foresee PHP[®] 4.0 Software Upgrade Instructions

Precautions:

- The user assumes full responsibility for the upgrade.
- Back up important data before upgrading
 - *Instructions for back up are found in the Foresee PHP User's Guide
- This software and its documentation is copyright Reichert.
- Reproducing or modifying all or part of this software or its documentation without permission is strictly prohibited.

Loading the Upgrade:

1. Download the Foresee PHP Software Version 4.0 to a USB Stick.
2. Plug the USB Stick into the USB Drive (Labeled **A** on Figure 1) located in the Removable Storage Device Compartment.



A


Figure 1. USB Stick plugged into the USB Drive

3. Switch the Foresee PHP to Microsoft[®] Windows mode
 - Instructions:** Switching to Microsoft Windows Environment from the Foresee PHP Main Menu screen
 - Click the **Options** button -> Click the **Switch to Windows Button**
 - The Foresee PHP will shut down and reboot into Windows mode.

4. Open **My Computer** to access your USB Drive
 - Go to **Start** → **My Computer**



5. Under My Computer, open the **Removable Disk (D:)** drive.

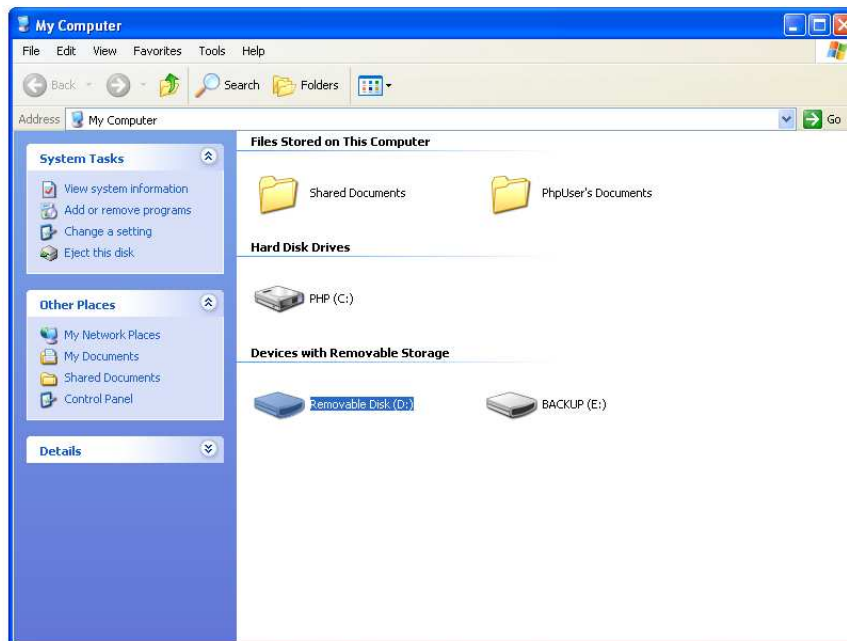


6. Double-click the **ForeseePHP4—Setup.exe** software icon.




ForeseePHP40-Setup.exe
Setup.exe
Macrovision Corporation

7. The **Foresee-PHP – InstallShield Wizard** window will appear.
- Click **Next >**

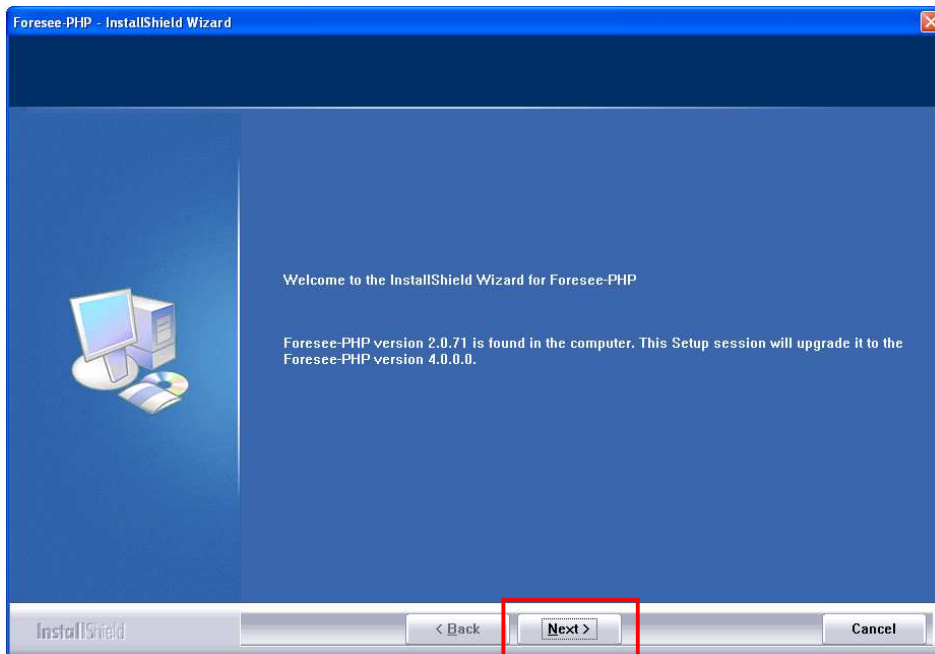


8. Choose the destination folder for the setup files.
- Click **Next >**

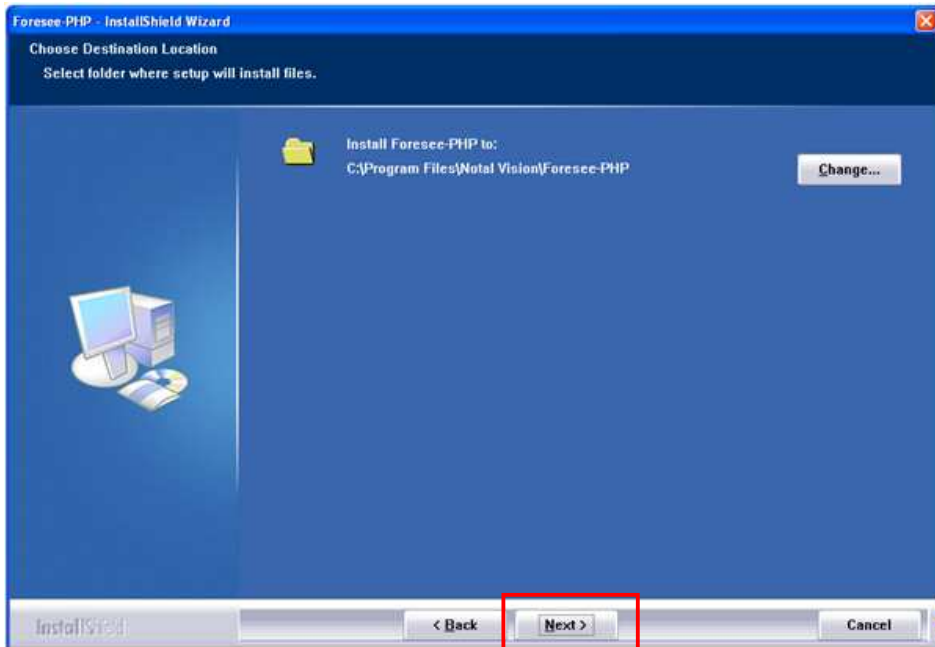


9. The Foresee PHP 4.0 software installation will automatically begin.


- A blue progress bar is presented during the automated installation

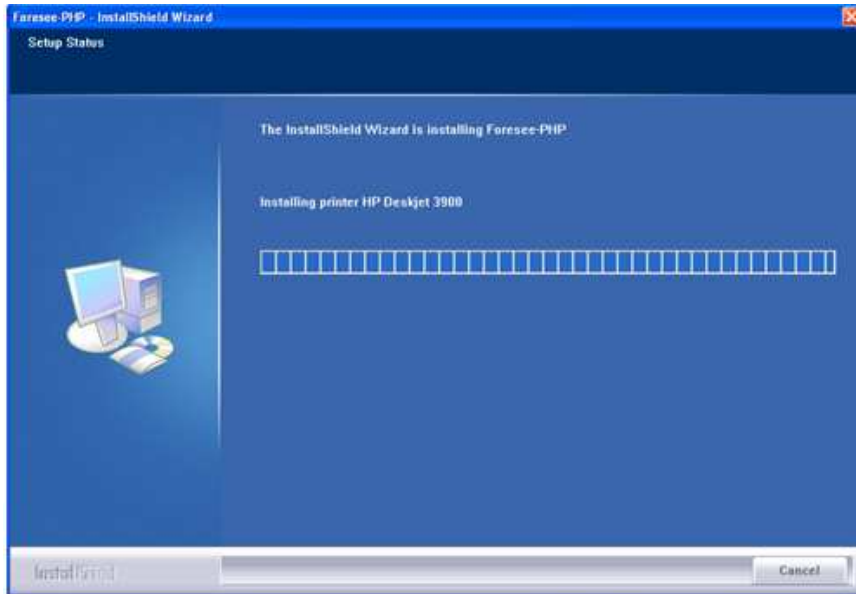

10. Once the InstallShield Wizard is complete, click **Finish**.

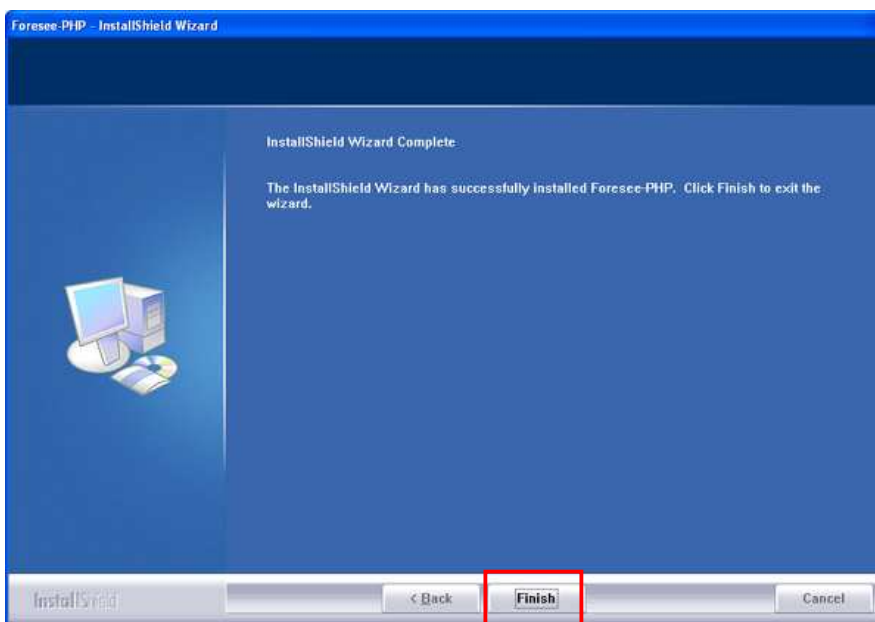

11. You will return to the Foresee PHP Control Panel.
- Click **Switch to the Foresee PHP**.
 - A window will pop up and read:
“The system will shut down Windows and restart the device in Foresee PHP mode. Do you want to continue?”
 - Click **YES** to return to Foresee PHP mode

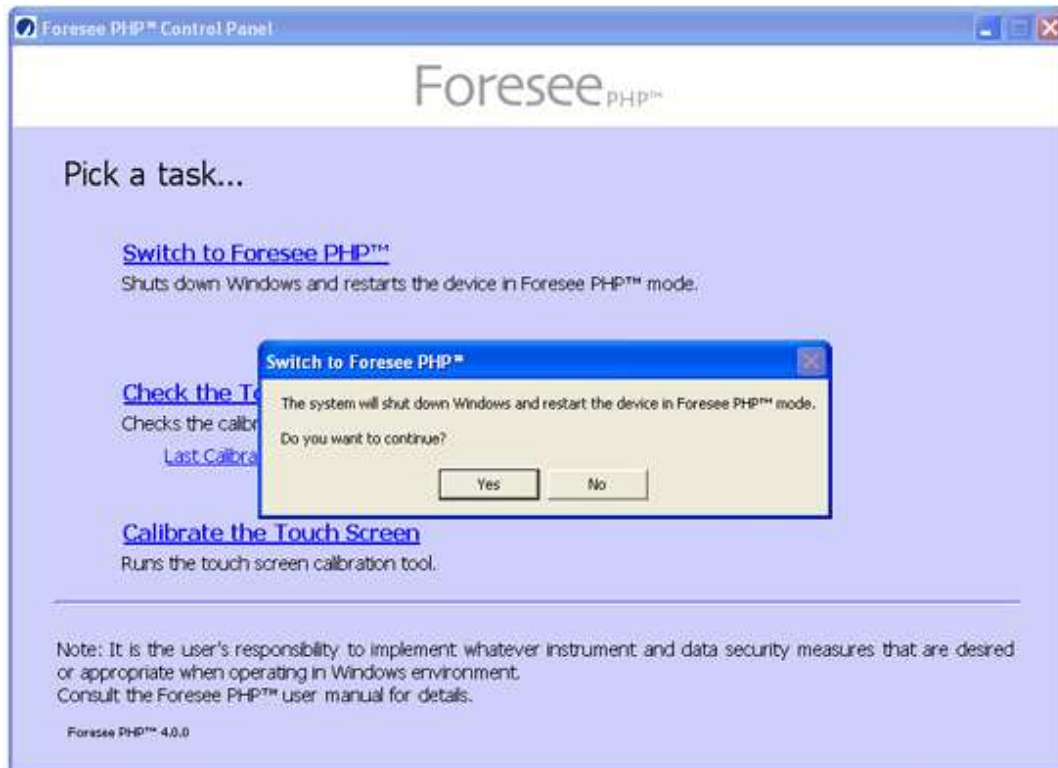

Reichert Technologies
3362 Walden Ave
Depew, NY 14043 USA
Toll Free: 888-849-8955
Phone: 716-686-4500
Fax: 716-686-4545
Email: info@reichert.com
www.reichert.com

Created: September 2010

