


# CATS™ TONOMETER PRISM CENTRATION REFERENCE GUIDE

REVIEW PRIOR TO USE




- CATS™ Tonometer Prism precise centration feature appears different from GAT.
- Decentration is indicated by distorted mires. Adjust slit-lamp to center mires.
- Mires will appear normal and round when the CATS™ Tonometer Prism is centered ensuring accurate IOP measurement.

CENTERED

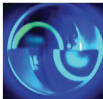

NOT CENTERED: DISTORTION

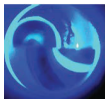

For support: 1-888-849-8955 or visit [reichert.com/cats](http://reichert.com/cats)

SCAN HERE OR VISIT  
[REICHERT.COM/CATS](http://REICHERT.COM/CATS) TO  
VIEW THE CENTRATION  
REFERENCE VIDEO



CAT04002 B