

TONO-PEN® XL

SERVICE SYMPTOM

Cleaning of the TONO-PEN XL required.

DESCRIPTION

When the TONO-PEN XL requires cleaning between patients as mandated by certain institutions.

CORRECTIVE PROCEDURE

CLEANING OF THE HANDLE

The TONO-PEN XL handle may be wiped down with a paper towel or soft cloth lightly moistened with alcohol.



Cleaning XL Handle

CAUTION: BE CAREFUL NOT TO USE EXCESS ALCOHOL ON OR AROUND THE LCD SCREEN, AS THIS MAY CAUSE DAMAGE TO THE SCREEN.

CLEANING OF THE TIP

There should always be a clean Ocu-Film® tip cover on the TONO-PEN XL, even when stored in its case. This will ensure that the tip stays clean. If accidental contamination of the tip occurs, and the tip requires cleaning, you may wipe down the tip with a paper towel or soft cloth lightly dampened with alcohol.



Cleaning XL Tip

CAUTION: BE CAREFUL NOT TO OVER MOISTEN THE PAPER TOWEL OR CLOTH. IF ANY EXCESS LIQUID GETS INSIDE THE PEN TIP, IT MAY DAMAGE THE UNIT, CAUSING PREMATURE FAILURE.

TONO-PEN® AVIA

SERVICE SYMPTOM

Cleaning of the TONO-PEN AVIA required.

DESCRIPTION

When the TONO-PEN AVIA requires cleaning between patients as mandated by certain institutions.

CORRECTIVE PROCEDURE

CLEANING OF THE HANDLE

The TONO-PEN AVIA handle may be wiped down with a paper towel or soft cloth lightly moistened with alcohol.



Cleaning AVIA Handle

CAUTION: BE CAREFUL NOT TO USE EXCESS ALCOHOL ON OR AROUND THE LCD SCREENS, AS THIS MAY CAUSE DAMAGE TO THE SCREENS.

CLEANING OF THE TIP

There should always be a clean Ocu-Film® tip cover on the TONO-PEN AVIA, even when stored in its case. This will ensure that the tip stays clean. If accidental contamination of the tip occurs, and the tip requires cleaning, you may wipe down the tip with a paper towel or soft cloth lightly dampened with alcohol.



Cleaning AVIA Tip

CAUTION: BE CAREFUL NOT TO OVER MOISTEN THE PAPER TOWEL OR CLOTH. IF ANY EXCESS LIQUID GETS INSIDE THE PEN TIP, IT MAY DAMAGE THE UNIT, CAUSING PREMATURE FAILURE.